TD WALKER

(FR) TD WALKER / TD MC WALKER

Botte de marche

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(GB) TD WALKER / TD MC WALKER

Walking boot

- Aluminum uprights are fixed at 90 degrees and can be contoured to the leg.
- Minimum dorsal flexion of metatarsals.
- Curved, anti-skid sole allows patient to walk naturally and comfortably.

Indications

The TD Walker boot is prescribed following a variety of foot, toe and ankle injuries or operations, in particular mild, moderate or severe ankle sprains and stable fractures of the foot or ankle.

Contraindications

- Not intended for unstable fractures or for fractures of the proximal tibia or fibula.
- Do not apply the product in direct contact with broken skin.

Precautions

- Follow the advice of the professional who prescribed or supplied the product.
- Consult your professional if discomfort occurs.
- Store at room temperature, preferably in its original packaging.
- For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions

- See product label.
- Fasten Velcro tabs before washing.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

Composition

See product label.

Fitting instructions

This walker comes packaged in a protective plastic wrap that includes a product code and sizing label. It is comprised of these components:

- Foam bootie that surrounds the foot and lower leg
- 2 forefoot straps and 2 calf straps
- Fitting guide and product information sheet

Putting on the boot:

1. Remove the walker from the large protective plastic bag and detach the foam bootie. DO NOT remove the plastic sleeves that are over the uprights. These plastic sleeves must remain over the uprights during the initial phases of the fitting procedure.

2. Open the leg and foot sections of the foam bootie by releasing the self-adhesive strip from the foam. Have the patient slide their foot and lower leg into the bootie.

3. Close the bootie, making sure that the foam is wrapped snugly around the patient's limb. The pre-sized bootie is designed with sewn edges, and you should not have to trim it to size (it can be cut, however, if needed).

4. Make sure that the plastic sleeves over the uprights are still completely covering the self-adhesive strip that is attached to the inside of the uprights. Have the patient place his or her foot in the aluminium shell between the two uprights. The foot should be positioned so the uprights are centered at midline on the side of the leg.

5. To increase stabilization, the uprights can be contoured to ensure an intimate alignment with the sides of the leg. There are also foam comfort pads mounted on the inside of the uprights (near the ankles) that can be moved to enhance comfort, or that can be removed to increase the medial-lateral width between the uprights.

6. When you are sure the uprights are properly contoured and in the proper alignment (centered at midline on the sides of the leg), remove the plastic sleeves from the uprights. The self-adhesive strip on the inside of the uprights will lock onto the foam bootie.

7. Tighten the two forefoot straps that secure the patient's foot in the boot. The forefoot straps should fit snugly. These straps can be adjusted by simply loosening or tightening the self-adhesive tabs. Any excessive material can also be trimmed off for a cleaner look.

8. Tighten the three straps that secure the calf between the uprights, starting with the strap closest to the ankle. These straps are in alternating directions to prevent rotation. The three straps should be tightened firmly.

9. Have the patient ambulate in the boot to ensure the walker is fitting and functioning properly.

Removing the boot — For removing the boot, the straps should be released and the self-adhesive strap that closes the bootie should be opened to allow the patient to step out of the boot **WITH THE BOOTIE STILL LOCKED IN POSITION IN THE TD WALKER SHELL**. This will enable the patient to easily reapply the bootie and boot with the proper leg alignment.

WARRANTY AND DISCLAIMER

This product is designed for short-term use (less than 180 days). It is not intended for reuse on a second patient. Townsend provides a limited 90 day warranty that offers repair or replacement of the boot or components including the foam wraps, inner sole pad and straps. Under normal use and conditions, it is unlikely that these components will wear out. Misuse, abuse or neglect including the placement of boots in the trunk of a car in temperatures that exceed 60 degrees can potentially void the warranty. This device is not guaranteed to prevent injuries or reduce or eliminate the risk of re-injury. This boot is not designed or intended to be worn by patients who weigh more than 113 kg.

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THUASNE SAS - 118-120 rue Marius Aufan - 92300 Levallois Perret France